Symphony[®]

*No. 1 choice
of hospitals
& mothers*

EN Instructions for use – hospital use | DA Brugsanvisning – hospitalsbrug
NO Bruksanvisning – for sykehus | SV Bruksanvisning – för sjukhusbruk
FI Käyttöohjeet – sairaalakäyttö | PL Instrukcja obsługi – użytkowanie w szpitalu

Thank you for choosing the Symphony breast pump. Breast milk is the natural choice for newborns, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and infants can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast pumps that meet breastfeeding mothers' needs. For mothers who insist on the best for their infants and themselves.

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Table of contents

4

1. Intended use/population/contraindication	5
2. Meaning of symbols	6
3. Important safety information	8
4. Product description	12
5. Cleaning	16
6. Preparing for pumping	26
7. Expression	32
8. Storing and thawing breast milk	42
9. Troubleshooting and error codes	43
10. Warranty and maintenance/disposal	45
11. Accessories list	47
12. Cleaning – country-specific instructions	280
13. EMC/technical description	283
14. Technical specifications	289

1. Intended use / population / contraindication

The intended use and indications of breast pumps

Breast pumps are intended to be used by lactating women in the hospital or home environment for expressing breast milk. Breast pumps are indicated to alleviate the symptoms of engorgement by expressing milk. In cases of mastitis, breast pumps support the healing process by removing breast milk from the affected breast. Breast pumps are indicated to relieve sore and cracked nipples and to bring out flat or inverted nipples. Also, breast pumps enable mothers to provide breast milk for babies who cannot feed directly at the breast for a variety of reasons, such as latch-on problems, cleft palate or premature babies.

Intended patient population / user

Adolescent paediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breast pump to express and store their breast milk once they have returned to work, if they are travelling, or are otherwise separated from their babies. Women are often delighted to learn that a breast pump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the sucking of a nursing baby.

Contraindications

There are no known contraindications for the Symphony* breast pump.

Product description

The Symphony multi-user breast pump is provided with the Symphony PLUS program card, which generates the pump suction patterns. The program card contains two pump programs. The goal of the INITIATE program is to initiate milk production with Medela's initiation technology. The MAINTAIN program is designed to build and maintain milk production with the 2-Phase Expression technology.

In some cases, the Symphony breast pump is provided with the Standard program card to generate the pump suction patterns. This program is designed to build and maintain milk production with 2-Phase Expression technology. While the Standard card can be used to initiate lactation, Medela recommends using the INITIATE program of the Symphony PLUS program card.

The 2-Phase Expression technology mimics the baby's natural sucking rhythm. Short, stimulating cycles in the stimulation phase are followed by longer ones in the expression phase.

2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury to the user or damage to the breast pump. There are several warning symbols/word combinations. The symbol/word combinations stand for:

Warning

Can lead to serious injury or death.

Note

Can lead to material damage.

Caution

Can lead to minor injury.

Info

Useful or important information that is not related to safety.

Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a cardboard package.

This symbol indicates that the device should be kept away from sunlight.

This symbol indicates a fragile device that should be handled with care.

This symbol indicates the temperature limits for operation, transport and storage.

This symbol indicates the humidity limits for operation, transport and storage.

This symbol indicates the atmospheric pressure limits for operation, transport and storage.

This symbol indicates that the device should be kept dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

This symbol indicates unique GSI Global Trade Item Numbers (GTIN).

Symbols on the device

This symbol indicates that the instructions for use should be followed.

This symbol indicates compliance with the essential requirements of Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

0123

This symbol indicates the manufacturer.

This symbol indicates that the device should not be disposed of with unsorted municipal waste (for EU only).

This symbol indicates compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates a type BF applied part.

This symbol indicates the manufacturer's serial number for the device.

This symbol indicates the manufacturer's catalogue number for the device.

IP21

This symbol indicates protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates a class II protection against electric shock with double or reinforced insulation.

This symbol indicates the fuse (to identify fuse boxes or their location).

This symbol indicates that the fuse box is safety tested.

This symbol indicates where to open the cover of the mains socket.

On/off button

Let-down button

Vacuum adjustment knob

Symbols on the display

Vacuum level indicator, stimulation phase – Standard program card and Symphony PLUS program card (MAINTAIN program)

Vacuum level indicator, expression phase – Standard program card and Symphony PLUS program card (MAINTAIN program)

Vacuum level indicator – Symphony PLUS program card (INITIATE program)

3. Important safety information

8

3.1 Warnings

Failure to follow the instructions/safety information below can lead to danger from the device. Subject to technical changes.

READ ALL INSTRUCTIONS BEFORE USE

WARNING: To reduce the risk of electrocution:

- I Keep the device dry. Never immerse in water or other liquids!
- I Do not spray or pour liquid directly onto the pump unit.
- I Do not place or store the device where it can fall or be pulled into a sink or tub.
- I Do not use the Symphony breast pump while bathing or showering.
- I Do not reach for any electrical device if it has fallen into water. Unplug immediately.
- I Always unplug the Symphony breast pump immediately after use.

WARNING: To avoid fire, electrocution or serious burns:

- I The Symphony breast pump is not heat-resistant: keep away from radiators and open flames.
- I Do not expose the motor unit to direct sunlight.
- I Do not operate the Symphony breast pump where aerosol (spray) products are being used or where oxygen is being administered.
- I Do not use the device in the presence of a flammable anaesthetic mixture with air, or with oxygen or nitrous oxide.
- I Do not thaw frozen breast milk, or do not warm breast milk in a microwave or in a pan of boiling water. Microwaving can cause severe burns to the baby's mouth from hot spots that develop in the milk during microwaving. Microwaving can also change the composition of breast milk.
- I Use only the power cord that comes with the Symphony breast pump. Make sure the voltage of the power cord is compatible with the power source.
- I Never operate an electrical device if it has a damaged power cord, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- I Always inspect power cord wires prior to use for damage or exposed wire. If any damage is found, immediately stop using the power cord. Contact the local Medela representative.

- | The Symphony breast pump should never be left unattended when plugged into a power source.
- | Separation from the mains is only assured by disconnecting the power cord from the wall socket.
- | Position the pump unit such that it is easy to disconnect the power cord from the wall socket.
- | Keep the power cord away from heated surfaces.

 WARNING: To avoid health risk and reduce risk of injury:

- | Reprocess all reusable parts that come in contact with the breast and breast milk prior to first use as shown in chapter 5.2.
- | Clean parts that come in contact with the breast and breast milk immediately after use.
- | If breast milk overflow occurs, reprocess according to the instructions in chapter 5.2.
- | The pump sets are personal care items intended for use by a single user only. They should not be shared between mothers without appropriate reprocessing to minimise the risk of cross-contamination.
- | Do not continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- | If problems or pain occur, consult a lactation specialist.
- | Never use while pregnant, unless prescribed/adviced by a physician, as pumping can induce labour.
- | For mothers who are infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV): pumping breast milk will not reduce or remove the risk of transmitting the virus to the baby through breast milk.
- | Inspect all components of the Symphony breast pump as well as all parts of the pump set(s) before each use.
- | Never use a device, components or parts that are damaged.
- | Never use a device, components or parts if dirt, mould or other contamination is visible.
- | Replace mouldy, damaged or worn parts.
- | There are no user-serviceable parts inside the Symphony pump unit. Repairs must be performed only by an authorised service agency. Do not repair the parts yourself!
No modifications to the device are permitted.
- | Use the Symphony breast pump only for its intended use as described in this manual.

3. Important safety information

10

WARNING: To avoid health risk and reduce risk of injury:

- I Use Medela original accessories only.
- I Do not use attachments not recommended by Medela, as this may cause hazards.
- I Do not modify any parts of the pump or the pump set.
- I Do not use the Symphony breast pump while sleeping or overly drowsy.
- I Do not use the Symphony breast pump while operating a motor vehicle.
- I Do not drive whilst hands-free pumping.
- I Never drop or insert any object into any opening or tubing.
- I Do not use outdoors.
- I Supervision is necessary when the Symphony breast pump is used in the vicinity of children.
- I Portable and mobile radio frequency communications equipment can affect the Symphony breast pump.

Important

- I **Plastic bottles and component parts become brittle when frozen and may break when dropped.**
- I **Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.**
- I **Take appropriate care in handling bottles and components.**
- I **Do not use the breast milk if bottles or components become damaged.**

3.2 Cautions

Failure to follow the instructions/safety information below can lead to minor injury. Subject to technical changes.

CAUTION: To avoid risk of contamination:

- I Wash hands thoroughly with soap and water before touching the breast pump, the pump set(s) and breasts. Avoid touching the inside of containers or lids.
- I Only use drinking-quality tap or bottled water for cleaning.
- I Do not store wet or damp parts as mould may develop.

⚠ CAUTION: To avoid health risk and reduce risk of injury:

- I Pump only with the breast pump in an upright position.
- I Pressing shields too firmly onto the breasts may affect the milk flow.
- I Do not express with a vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma, may decrease the milk output.
- I Do not attempt to remove the breast shield from the breast while pumping. Turn the breast pump off and break the seal between the breast and the breast shield with the finger, then remove the breast shield from the breast.
- I If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with the finger and remove the breast shield from the breast.
- I Do not shorten the tubing between the breast shield and the pump.
- I Do not use abrasive washing-up liquids/detergents when cleaning the breast pump, pump components or pump set parts.
- I Never put the breast pump unit in water or in a steriliser, as this can cause permanent damage to the breast pump.
- I Do not use the Medela Symphony pump sets with a non-compatible breast pump.
- I Do not use the pump sets for any other purpose than expressing breast milk.

⚠ CAUTION: To reduce risk of electrocution:

- I Plug the power cord into the breast pump first and then into the wall socket.
- I Unplug the breast pump before cleaning.
- I The Symphony breast pump needs special precautions regarding EMC (Electro Magnetic Compatibility). Install and put into service according to the EMC information provided in these instructions for use.

4. Product description

12

4.1 Symphony hospital breast pump and accessories

The Symphony multi-user breast pump is provided with the Symphony PLUS program card, which generates the pump suction patterns. The program card contains the INITIATE program with Medela's initiation technology and the MAINTAIN program with the 2-Phase Expression technology.

It is possible that the Symphony Standard program card is installed in your Symphony breast pump. This program card contains the Standard 2.0 program with 2-Phase Expression technology.

Required accessories*

The Symphony breast pump should be operated with Medela pump sets and milk bottles. The pump sets and the bottles are available as reusable or as disposable products.

Reusable products:

I Symphony Hospital Multibox (20 reusable pump sets in size M, L or XL).

Disposable products:

Medela's disposable products are intended for single-use or one-day use in hospitals, without a need for cleaning prior to first use:

Medela's **Ready-to-Use products** are hygienically safe to use without prior cleaning, for all hospitals where the use of non-sterile products is allowed.

Medela's **sterile products** are EO sterile and fulfill the EN/ISO standards for medical devices ISO 11607-1, ISO 11607-2, ISO 11135-1.

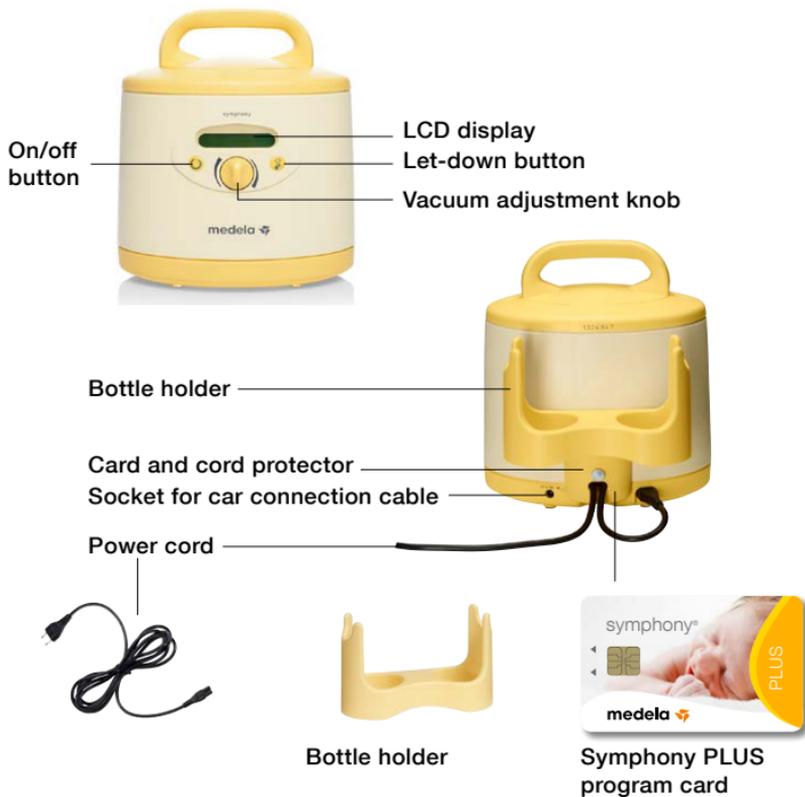
Available disposable products:

- I Symphony One-Day Pump Set with PersonalFit breast shield (size M, L or XL).
- I Sterile Single-Use Symphony Pump Set (size M, L or XL).
- I Ready-to-Use One-Day Symphony Pump Set (size M, L or XL).
- I Colostrum Container (35mL)
- I Sterile Single-Use Bottle (available sizes: 80, 150 and 250 mL)
- I Ready-to-Use Single-Use Bottle (available sizes: 80, 150 and 250 mL)

Optional accessories*

Mobile stand for the breast pump, hard case for the pump unit, card and cord protector.

4.2 Symphony hospital breast pump – Product parts*


* See chapter 11 for ordering information for accessories

4. Product description

14

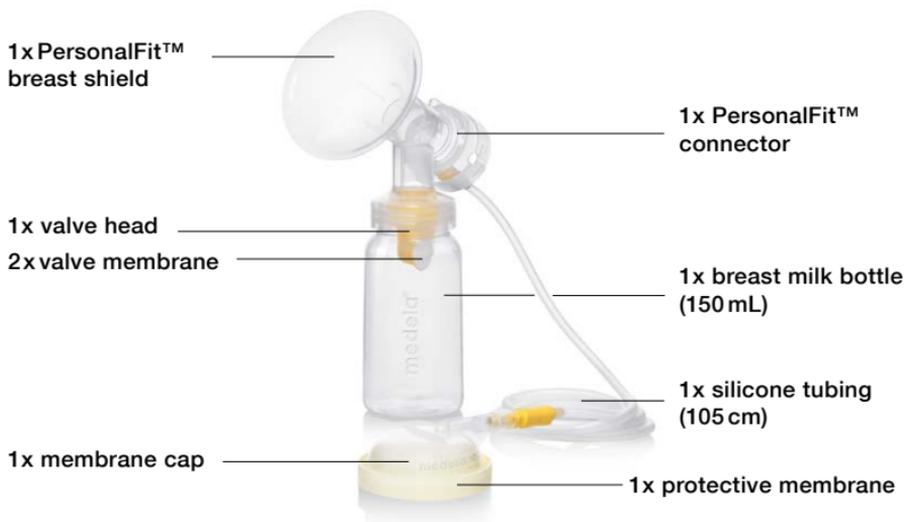4.3 Reusable pump set – Product parts*

1x protective cap

1x disc

1x lid with opening

* See chapter 11 for ordering information

4.4 Disposable pump set – Product parts*

1x breast shield

1x connector

1x membrane cap

1x membrane

1x PVC tubing

1x protective membrane

colostrum
container
35 mL

disposable
bottle 80 mL

disposable
bottle 150 mL

disposable
bottle 250 mL

* See chapter 11 for ordering information

5. Cleaning

16

Caution

- | Unplug the breast pump before cleaning.
- | These recommendations are not substitutes for official hospital procedures that may differ among institutions. Refer to hospital policy for cleaning of shared capital equipment.
- | Surface disinfectant agents for medical devices will not cause mechanical damage to the Symphony.
- | Follow the instructions provided by the supplier of the surface disinfectant agent.
- | Provide personal protection for operators in accordance with the supplier's instructions and safety data sheets.
- | Check the suitability of the surface disinfectant agents for medical devices by reference to the supplier's information and/or physical testing.

5.1 Breast pump – Cleaning and disinfection

Clean and disinfect the breast pump as needed.

To **clean** the breast pump, wipe the pump housing with a clean, damp (not wet) cloth. Drinking-quality water and a basic washing-up liquid or non-abrasive detergent may be used.

5.2 Cleaning pump sets – Overview

To **disinfect** the pump, wipe the pump housing with a clean, damp (not wet) cloth that is impregnated with a surface disinfectant agent for medical devices. Medela recommends using a surface disinfectant agent that is alcohol based.

The table on the following pages shows when to clean, disinfect and sterilise the bottles, pump sets and the tubing. These are general instructions. For country-specific regulations and guidelines, see chapter 12. The listed automated reprocessing procedures are tested, validated and recommended by Medela. In the case of automated cleaning, an internal validation of the preferred procedure by the hospital is necessary, too. In the event of conflicting national cleaning and sterilisation requirements, these shall prevail over the Medela recommendations.

5. Cleaning

5.2 Cleaning pump sets – Overview

	Disposable bottles ¹⁾ and disposable pump sets (breast shield, connector, membrane, PVC tubing, membrane cap, protective membrane)	
	② Single-use pump set and single-use bottles	One-day pump sets
Before first use	No cleaning needed	No cleaning needed
After every use ²⁾	Dispose of after use (§5.3)	I Manual cleaning (§5.4) Note: Tubing, membrane cap and protective membrane must not be cleaned.
Once a day	n/a	Dispose of after a maximum of 8 pumping sessions within 24 hours (§5.6).
Before passing on to the next mother	n/a	n/a

¹⁾ All disposable bottles (sterile and Ready-to-Use) are single-use products. They must not be reprocessed.

²⁾ As long as the pump sets stay with the mother.

	Reusable bottles and reusable pump sets (breast shield, connector, valve head, valve membrane, membrane cap, protective membrane)	Reusable pump sets (silicone tubing) 
Before first use	<ul style="list-style-type: none"> I Automated cleaning and disinfection (§5.8) or I Automated cleaning and disinfection (§5.8) and sterilisation⁴⁾ (§5.9) 	Wipe disinfection (§5.7)
After every use²⁾	<ul style="list-style-type: none"> I Manual cleaning (§5.7) or automated cleaning and disinfection (§5.8) I Sterilisation⁴⁾ (§5.9) 	No cleaning needed ³⁾
Once a day²⁾	<ul style="list-style-type: none"> I Manual cleaning (§5.7) or automated cleaning and disinfection (§5.8) I Sterilisation⁴⁾ (§5.9) 	No cleaning needed ³⁾
Before passing on to the next mother	<ul style="list-style-type: none"> I Automated cleaning and disinfection (§5.8) I Sterilisation⁴⁾ (§5.9) 	Wipe disinfection ³⁾ (§5.7)

²⁾ As long as the pump sets stay with the mother.

³⁾ If there is milk, condensation or visible residues in the tubing, the tubing must be disposed of and replaced. Medela recommends that the tubing is disposed of and replaced after max. 6 months.

⁴⁾ If required by national, regional or in-house regulations.

5. Cleaning

20

Warning

- | The sterile single-use pump set is a **single-use** product and may not be reused or reprocessed.
- | The one-day pump set is a **single-user** product. It is not possible to disinfect and autoclave the pump set for safe re-use between multiple users. Use by more than one person may present a health risk and could cause cross-contamination.
- | The disposable bottles (sterile and Ready-to-Use) are **single-use** products that may not be reused or reprocessed.
- | Take apart and wash all parts that come into contact with the breast and breast milk immediately after use to avoid milk residues drying up and to prevent growth of bacteria.

Caution

- | Only use drinking-quality water for cleaning.

Note

- | The pump sets require no maintenance.
- | Take care not to damage parts of the one-day pump set during cleaning.
- | You can use a one-day pump set for up to 8 pumping sessions within 24 hours. For details, see the pump set's instructions.

5.3 Disposable single-use pump sets and bottles – Cleaning

The sterile pump set and disposable bottles (sterile and Ready-to-Use) are **single-use** products that may not be reused or reprocessed. Dispose of the pump set and bottle after use. Use a new pump set and bottle for each pumping session.

3 Drying / storage

Dry with a clean cloth or leave to dry on a clean cloth. Put the clean parts in a clean storage bag or a clean environment. It is important that all residual moisture dries. Do not store parts in an airtight container/bag.

5.4 Disposable one-day pump set – Cleaning

Check whether there is milk in the tubing. If so, replace the pump set. If not, disassemble the pump set into its individual parts (breast shield, connector, membrane, membrane cap, protective membrane and tubing). You must also remove the white valve from the connector.

→ Note that the membrane cap, protective membrane and tubing must not be cleaned.

Rinse the breast shield, the connector and the white valve with cold, clear drinking-quality water (approx. 20 °C) to remove proteins. Clean these parts with plenty of warm, soapy water (approx. 30 °C) to wash off fat. Use a commercially available washing-up liquid, preferably without artificial fragrances and colouring (5–15% anionic tenside, nonionic tenside, pH neutral). Rinse the parts with cold, clear drinking-quality water (approx. 20 °C).

5.5 Disposable one-day pump set – Disinfection and sterilisation

Do not disinfect or sterilise the disposable one-day pump set.

5.6 Disposable one-day pump set – Disposal

Dispose of the disposable one-day pump set after a maximum of eight pumping sessions within 24 hours.

5. Cleaning

22

Warning

- | Use by more than one mother without adequate reprocessing may be a health risk and could cause cross-contamination.
- | Clean parts that come into contact with the breast and breast milk immediately after use.
- | Replace mouldy, damaged or worn parts.

Caution

- | Only use drinking-quality water for cleaning.

Note

- | Only sterilise clean and intact parts. Do not stack parts on top of one another. To avoid damage, do not apply external pressure.
- | Medela does not define a maximum number of uses appropriate for a reusable pump set or its individual parts, with exception of the tubing. The useful life of a pump set depends on many factors including the duration of each use and handling between uses. Careful inspection and a functional test of the pump set (parts) before use is the best method of determining the end of life of the device. Medela recommends that the silicone tubing is disposed of and replaced after max. 6 months.

5.7 Reusable pump set – Manual cleaning

- 1
- Disassemble the reusable pump set into its individual parts (bottle, breast shield, connector, valve head, valve membrane, membrane cap, protective membrane and tubing). Gently remove the white membrane from the yellow valve head.
- Check all parts for damage. Replace damaged or worn parts.

4

Drying/storage

- Dry with a clean cloth or leave to dry on a clean cloth.
- Put the clean parts in a clean storage bag or a clean environment. It is important that all residual moisture dries. Do not store parts in an airtight container/bag.

2

I No milk, condensation or visible residues in the tubing

External wipe disinfection is recommended before first use or before passing on to the next mother. Use alcohol-based disinfectants that contain no additives and dry without leaving residue. Medela recommends that the tubing is disposed of and replaced after max. 6 months.

I With milk, condensation or visible residues

Dispose of the tubing and replace.

3

Rinse all parts, except for the tubing, with cold, clear drinking-quality water (approx. 20°C) to remove proteins. Clean these parts with plenty of warm, soapy water (approx. 30°C) to wash off fat. Use a commercially available washing-up liquid, preferably without artificial fragrances and colouring (5–15% anionic tenside, nonionic tenside, pH neutral). Rinse the parts with cold, clear, drinking-quality water (approx. 20°C).

5. Cleaning

24

Warning

- | Use by more than one mother without adequate reprocessing may be a health risk and could cause cross-contamination.
- | If there is no milk, condensation or visible residues in the tubing, there is no need for automated cleaning and disinfection or sterilisation. Medela recommends that the tubing is disposed of and replaced after max. 6 months.

If there is milk, condensation or visible residues in the tubing, dispose of and replace.

Caution

- | Only use drinking-quality water for cleaning.

Note


- | Only sterilise clean and intact parts. Do not stack parts on top of one another. To avoid damage, do not apply external pressure.

5.8 Reusable pump set – Automated cleaning and disinfection

If not done yet, disassemble the reusable pump set into its individual parts (bottle, breast shield, connector, valve head, valve membrane, membrane cap, protective membrane and tubing). Gently remove the white membrane from the yellow valve head. Check all parts for damage. Replace damaged or worn parts.

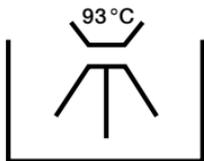2 3 Storage

For cleaning of the tubing see chapter 5.7. Clean and disinfect the remaining parts in a mechanical washer-disinfector. Use appropriate inserts for the disinfection of small parts. Use a common cleaning detergent for washer-disinfectors (NTA-, phosphate-, chlorine- and tenside-free, alkaline with pH level 9,5–12). Disinfect for five minutes at a temperature of 93 °C. Select an appropriate program, according to the above instructions, that includes drying*.

Put the clean parts in a clean storage bag or a clean environment. It is important that all residual moisture dries. Do not store parts in an airtight container/bag.

*If the appropriate program does not include drying, dry the clean parts with a clean cloth or leave to dry on a clean cloth.

5.9 Reusable pump set – Sterilisation

If national, regional or in-house regulations require sterilisation, do as follows:

- I Check that the parts of the pump set are clean, dry and intact.
- I Wrap all parts, including the membrane and the membrane cap, in appropriate sterile packaging, such as a paper film.
- I Sterilise the product in the autoclave at 134 °C for 3 minutes. To avoid damage, do not stack or apply external pressure.
- I Leave the product parts in the sterile packaging and store them in a clean, dry and dust-free environment. It is important that all residual moisture dries. Do not store in an airtight container/bag.

6. Preparing for pumping

26

Warning

- | Only use the power cord that comes with the Symphony.
- | Make sure the voltage of the power cord is compatible with the power source.

Info

Optional for pump with battery:

- | The built-in battery must be charged for 12 hours on receipt of the device.
- | The pump can be used during charging.
- | If the pump is to remain unused for a long time, charge the battery every two months (for 12 hours each time).
- | The optimum service life of the battery is achieved if it is not fully discharged on a regular basis or if the pump is charged while not in use.

Pumping time (fully charged battery)	approx. 60 min.
Charging time (for full charge)	12 hours

6.1 Installation of the program card

The program card must be inserted in the card compartment on the back of the breast pump. For more information on the use of the card and the selection of the appropriate program, see chapter 7.2.

To remove, unscrew the card and cord protector and slide out of the guide rail.

6.2 Installation of the card and cord protector (optional)

1

Insert the cord into the card and cord protector in a loop around the pin.

→ Make sure there is enough play to plug the cord in.

2

Push the card and cord protector over the guide rail on the back until it stops and tighten the screw.

6.3 Mains operation

1

Insert the power cord into the power socket on the back of the pump.

2

Plug the power cord into the mains socket.

6. Preparing for pumping

28

Caution

Wash hands thoroughly (at least one minute) with soap and water before touching the breast, the clean pump and the clean pump set parts. Dry your hands with a fresh towel or a single-use paper towel.

Note

- Use Medela original accessories only.
- The disposable pump set does not need cleaning prior to first use.
- The reusable pump set needs cleaning prior to first use. See the cleaning overview in chapter 5.2.
- Check pump set parts for wear or damage before use. Replace damaged or worn parts.
- Check that the transparent cap is attached to the protective membrane correctly.
- All parts must be completely dry before use.

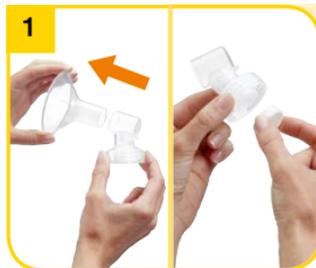
Info

Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.

* For more detailed information on the use of the pump set, see the pump set's instructions for use.

6.4 Assembling the disposable pump set*

Open the packaging of the disposable pump set. Push the connector onto the breast shield. Then push the white valve onto the connector (if not yet assembled).

6.5 Assembling the reusable pump set*

Carefully push the breast shield onto the connector. Choose a breast shield size that suits your needs.

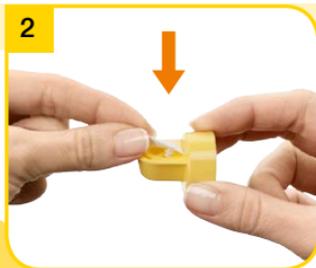
Screw a disposable bottle onto the connector. Choose a bottle that suits your needs. Sizes available are 35, 80, 150 and 250 mL. Push the open end of the tubing over the protruding part at the back of the connector.

Continue with chapter 6.6

Check that the transparent cap at the other end of the tubing is covered with the protective membrane. If not, place the membrane with the round side up onto a flat surface. Push the cap into the membrane until it engages.

Place the white membrane in a horizontal position on the valve head. Push the knob on the membrane's back into the smallest hole of the valve head until it engages.

Push the valve head and membrane onto the connector.
→ Position the valve head sideways.

6. Preparing for pumping

30

i Info

Always close the lid when pumping. The lid keeps the membrane cap pushed down to prevent a loss of vacuum that could occur if the membrane cap came off.

4

Screw a Medela bottle onto the connector. Choose a size that suits your needs.

6.6 Connecting the pump set to the pump housing

1

Open the breast pump lid: Press the oval button on the top of the Symphony and lift the handle.

5

Insert the tubing into the corresponding opening in the connector.

6

Continue with chapter 6.6

Place the membrane with the round side up onto a flat surface. Push the cap into the membrane until it engages. Insert the tubing adapter into the opening in the membrane cap.

2

Position the membrane cap at the end of the tubing firmly onto the membrane in the housing, to form a seal. Centre the tubing connector in the recess in the pump housing.

→ Make sure that the membrane cap engages and lies flush to the surface.

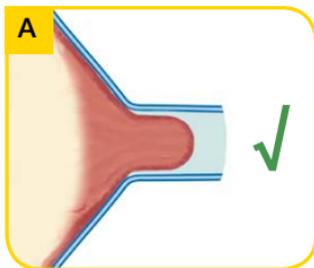
3


Close the lid.

7. Expression


32

This breast shield has an optimum fit.

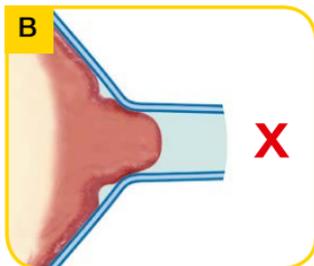This breast shield is too small.

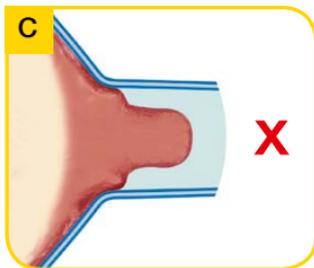This breast shield is too large.

7.1 Selecting the breast shield

A correctly fitted breast shield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

To verify whether the breast shield has a correct fit:

1. Centre the nipple in the breast shield tunnel.
2. Turn on the pump and check the following points.

! Is the nipple moving freely in the tunnel?

! Is minimal or no areolar tissue being pulled into the tunnel of the breast shield?

! Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?

! Do you feel the breast emptying all over?

! Is the nipple pain free?

If the answer to any of above questions is "No", try a larger or smaller size of the PersonalFit breast shield.

PersonalFit breast shield sizes available are:
S (21 mm), M (24 mm), L (27 mm),
XL (30 mm) or XXL (36 mm).*

* See chapter 11 for ordering information

7.2 Selecting the right program*

The Symphony breast pump is installed with the Symphony PLUS program card, which contains two pump programs. The goal of the INITIATE program is to initiate milk production. The MAINTAIN program is designed to build and maintain milk production.

→ It is possible that the Symphony Standard program card is installed on your Symphony breast pump. The Standard program is optimised to build and maintain milk production after secretory activation has occurred. For operating this card, see §7.5. While the Standard Card can be used to initiate lactation, Medela recommends using the INITIATE program of the Symphony PLUS program card. See chapter 11 for ordering information.

When the Symphony PLUS card is installed:

Use the INITIATE program directly after birth:

- ! Until the mother has pumped at least 20mL in total in each of her last three pumping sessions, OR
- ! Throughout the first five days. If milk production has not started after five days, change to the MAINTAIN program.

Use the MAINTAIN program after secretory activation:

- ! Once the mother has pumped at least 20mL in total in each of her last three pumping sessions, OR
- ! Beginning on day six, whichever of the two options comes first.

7.3 Operation: INITIATE program

Press the On/off button  to switch on the breast pump. The display text shows "INITIATE press .

Press the "Let-down" button  **within 10 seconds**. The display text will change to "INITIATE running".

* For more detailed information on the program cards, see their instructions for use.

7. Expression

34

Note

Always disconnect the Symphony from the power source after expression.

Info

The program stores the last vacuum setting in the stimulation phase.

I for the MAINTAIN program:

Press the “Let-down” button  to revert from the expression phase to the stimulation phase if necessary. If milk has not begun to flow after two successive stimulation phases, the mother should take a break and try again in 15–30 minutes. Also, she could try breast massage and then pump using the expression phase.

If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Symphony will automatically switch off.

3

Set the comfort vacuum (see chapter 7.6). The bullets on the display show the vacuum level. The more bullets, the stronger the vacuum.

2

Set the comfort vacuum for the stimulation phase (see chapter 7.6). The drops on the display show the vacuum level. The more drops, the stronger the vacuum.

7.4 Operation: MAINTAIN program

The INITIATE program runs automatically for 15 minutes with stimulation, expression and pause phases. An acoustic signal indicates the end of the program. The display shows "Program complete". The pump switches off automatically.

→ It is important to complete the entire INITIATE program.

Press the On/Off button  to switch on the breast pump. The MAINTAIN program will automatically start with the stimulation phase after pressing the On/Off button.

The pump will automatically switch to the expression phase after two minutes.

→ If the milk starts to flow sooner, press the Let-down button  to change to the expression phase.

Set the comfort vacuum for the expression phase (see chapter 7.6). The bars on the display show the vacuum level. The more bars, the stronger the vacuum.

7. Expression

36

Note

Always disconnect the Symphony from the power source after expression.

Info

The program stores the last vacuum setting in the stimulation phase.

for the Standard program:

Press the “Let-down” button  to revert from the expression phase to the stimulation phase if necessary. If milk has not begun to flow after two successive stimulation phases, the mother should take a break and try again in 15–30 minutes. Also, she could try breast massage and then pump using the expression phase.

If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Symphony will automatically switch off.

5

Continue pumping until the breast feels well drained and soft all over (Medela recommends a pumping time of at least 15 minutes). Switch off the pump by pressing the On/off button .

3

The pump will automatically switch to the expression phase after two minutes.

→ If the milk starts to flow sooner, press the Let-down button  to change to the expression phase.

7.5 Operation: Standard program

1

Press the On/off button  to switch on the breast pump. The Standard program will automatically start with the stimulation phase after pressing the On/off button.

2

Set the comfort vacuum for the stimulation phase (see chapter 7.6). The drops on the display show the vacuum level. The more drops, the stronger the vacuum.

4

Set the comfort vacuum for the expression phase (see chapter 7.6). The bars on the display show the vacuum level. The more bars, the stronger the vacuum.

5

Continue pumping until the breast feels well drained and soft all over (Medela recommends a pumping time of at least 15 minutes). Switch off the pump by pressing the On/off button .

7. Expression

38

Caution

- I Wash hands thoroughly (at least one minute) with soap and water before touching the breast, the clean pump and the clean pump set parts. Dry your hands with a fresh towel or a single-use paper towel.
- I Contact the midwife or breast-feeding and lactation consultant if only minimal or no milk can be expressed or if expression is painful.

Note

- I Always disconnect the Symphony from the power source after expression.

Info

- I Do not kink the tubing during expression.

7.6 Set comfort vacuum

Set the comfort vacuum by rotating the vacuum adjustment knob. First rotate to the right to increase the vacuum until there is a slight discomfort. Then rotate to the left to reduce the vacuum until it is comfortable.

Switch on the breast pump with . Proceed with the program selection (see chapter 7.2). Operate the pump as is appropriate for the selected program (see chapters 7.3, 7.4 and 7.5).

7.7 Expression by single pumping

1 Prepare one pump set for expression. Place the membrane cap at the end of the tubing onto the membrane in the pump housing (see also chapter 6.6). Wipe the breast with a warm cloth (do not use alcohol).

2 For breast shield fitting instructions, see chapter 7.1

Place the breast shield on the breast so that the nipple is properly **centred** in the tunnel. Hold the breast shield onto the breast with the thumb and index finger. Support the breast with the palm of the hand.

4 When expression is finished, place the bottle in the stand or bottle holder. This is to prevent the bottle from tipping over.

5 Clean in accordance with chapter 5

Close the bottle with a lid or a lid and disc (depending on the bottle type). To store the breast milk, follow the instructions in chapter 8.

7. Expression

40

Caution

I Wash hands thoroughly (at least one minute) with soap and water before touching the breast, the clean pump and the clean pump set parts. Dry your hands with a fresh towel or a single-use paper towel.

Note

I Always disconnect the Symphony from the power source after expression.

Info

I **Double pumping** saves time and increases the energy content of the milk. Milk production may be increased and maintained long term.

I Do not kink the tubing during expression.

7.8 Expression by double pumping

Prepare two pump sets for expression. First place the membrane cap at the end of tubing one onto the membrane in the pump housing (see chapter 6.6). Then place the membrane of the second tubing onto the other membrane in the housing. Wipe the breast with a warm cloth (do not use alcohol).

Place the breast shield of the second pump set on the other breast so that the nipple is properly **centred** in the tunnel. Operate the pump as is appropriate for the selected program (see chapters 7.3, 7.4 and 7.5).

2

For breast shield fitting instructions, see chapter 7.1

Place the breast shield of one pump set on the first breast so that the nipple is properly **centred** in the tunnel. Hold the breast shield onto the breast with the thumb and index finger. Support the breast with the palm of the hand.

3

Switch on the breast pump with . Proceed with the program selection (see chapter 7.2).

5

When expression is finished, place the bottles in the stand or bottle holder. This is to prevent the bottles from tipping over.

6

Clean in accordance with chapter 5

Close the bottles with a lid or a lid and disc (depending on the bottle type). To store the breast milk, follow the instructions in chapter 8.

8. Storing and thawing breast milk

8.1 Storage

Storage guidelines for freshly expressed breast milk

	Room temperature 16 to 26 °C (60 to 78 °F)	Refrigerator 4 °C (39 °F) or lower	Freezer -18 to -20 °C (0 to -4 °F)	Thawed breast milk
Healthy term infant	≤ 6 hours	≤ 5 days	≤ 6 months ideal, maximum 12 months	Thawed to room temperature: use within max. 4h Thawed to fridge temperature: use within 24h
NICU infant	Use within 4 hours	Use within 4 days	Optimal: use within 3 months	Do not refreeze!

I Use of a medical grade refrigerator with temperature uniformity and constant temperature monitoring is recommended.

8.2 Freezing

I You can freeze expressed breast milk in milk bottles or “Pump & Save”* bags. Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.

I Label the bottles or “Pump & Save” bags with the date of expression and volume of milk expressed.

8.3 Thawing

Warning

Do not thaw frozen breast milk or do not warm breast milk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

I In order to preserve the components of the breast milk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Pump & Save” bag under warm water (max. 37 °C / 98.6 °F).

I Gently swirl the bottle or “Pump & Save” bag to mix any fat that has separated. Avoid shaking or stirring the milk.

I With the Calesca* bottle warmer, it is possible to warm breast milk in a gentle way.

Note: These storing and thawing breast milk guidelines are a recommendation. National and hospital internal guidelines and standards may deviate.

9. Troubleshooting and error codes

9.1 Troubleshooting

Problem	Solution
If the motor is not running	<p>Check whether mains power/battery is available.</p> <p>Check that the device is switched on.</p> <p>Check whether program card is inserted correctly in the card compartment on the back of the breast pump.</p>
If there is low or no suction	<p>Assemble the pump set in accordance with the instructions in chapter 6.4 or 6.5.</p> <p>Check whether all connections to the pump set and breast pump are secure.</p> <p>Express in accordance with chapter 7. Make sure that the breast shield forms a complete seal around the breast.</p> <p>The white valve membrane must lie flat against the yellow valve head.</p> <p>Check that the valve head and membrane are clean and not damaged.</p> <p>See chapter 6.4 or 6.5 and 6.6:</p> <ul style="list-style-type: none">I The ends of the tubing must be firmly connected to the connector and the opening in the membrane cap.I The protective membrane and membrane cap should be correctly assembled and the membrane cap firmly fitted to the breast pump membrane to form a seal.I The protective membrane must be intact.I Do not kink or trap the tubing during expression.I All components must be completely dry.
If there is an overflow	<ul style="list-style-type: none">I Place the pump set into the bottle holder.I Turn off the pump and unplug the power cord from the socket.I Use a damp (not wet) cloth to wipe the pump and membrane area.I If there is milk, condensation or visible residues in the tubing:<ul style="list-style-type: none">For disposable pump set: dispose of the pump set and replace with a new one.For reusable pump set: dispose of the silicone tubing and replace with a new one. Reprocess the remaining parts according to chapter 5.2I When components are completely dry, reassemble breast pump and accessories set.

9. Troubleshooting and error codes

9.2 Error codes

Error codes

The following describes error codes that may be displayed indicating an issue with the unit, potential causes, and steps to correct the issue.

	Potential issues	Actions
Invalid card	Program card is not a Symphony card	- Check if the correct program card is being used
	Program card is incorrectly inserted	- Check that card is positioned correctly
	Program card is damaged and not working	- Replace program card
No card	Program card is not inserted	- Insert program card
	Program card is not completely inserted	- Push card in all the way
Battery low	Battery needs charging soon, will beep every 20 seconds (15 minutes of pumping time remaining from when the indication started)	- Be sure power cord is plugged into the back of the pump
Motor overload	Motor is drawing too much current	- Contact the local Medela representative if this happens repeatedly
Error	Potential fault in electronics of gear control	- Contact the local Medela representative if this happens repeatedly
Power fault	Potential fault in electronics of power supply	- Contact the local Medela representative if this happens repeatedly

10. Warranty and maintenance/disposal

Warranty and maintenance

Three-year warranty for the unit from the date of purchase, excluding pump set and accessories. Six month warranty for the batteries. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorised persons. Routine checks, servicing and battery exchange may only be carried out by Medela authorised personnel.

The engineering of the Medela Symphony breast pump has been verified by independent test institutes for compliance with the EN/IEC 60601-1 standard. Copies of such certificates may be obtained on request. Because of the method of construction used, Medela does not expect electrical safety to be affected at any time during the life of the product – provided that it is used properly in accordance with the intended use as well as these instructions for use, and that the device is serviced and repaired at authorized service centre. There are no prescribed or recommended electrical safety checks or recurrent tests to be carried out.

Manufacturer's information provided in accordance with EN/IEC 62353:2014 "Medical electrical equipment – Recurrent test and test after repair of medical electrical equipment".

Besides the cleaning procedure(s) described in chapter 5 and battery charging for pumps with optional batteries as described in chapter 6, there is no further maintenance required. Besides the inspection procedures described in chapter 3, e.g. visual signs of damage on pump or power cord, pump dropped or dropped into water, there are no further inspections required. As pointed out in chapter 3, service and repair must be performed only by an authorized service agency (centre) in full accordance with the Symphony Service Manual using original Medela spare parts mentioned therein. The service manual, wiring diagrams, and descriptions are available upon request from Medela.

Medela deems that the final production testing replaces on-site testing of the device before putting into service.

The service life of the device is seven years, the internal batteries life included.
The service life of the accessories is two years.

10. Warranty and maintenance/disposal

46

Disposal

The Symphony breast pump is made of metals and plastics and should be disposed of in accordance with the European directives 2002/95/EC and 2002/96/EC. Additional, local guidelines must also be observed. With the AC/DC versions, the electronic components and the rechargeable battery must be disposed of separately, in accordance with the local regulations. Please take care that you dispose of the Symphony and its accessories in accordance with your local disposal guidelines.

User information for the disposal of electrical and electronic equipment

This symbol means that the electrical and electronic equipment must not be disposed of together with unsorted municipal waste. Correct disposal of this device protects and prevents possible damage to the environment or human health. For more information about the disposal contact the manufacturer, your local caregiver or healthcare provider. This symbol is only valid in the European Union. Please respect the relevant state laws and rules in your country for the disposal of electrical and electronic equipment.

11. Accessories list*

Symphony PLUS program card

Article number	Product
200.9186	Symphony PLUS program card English
200.9187	Symphony PLUS program card German
200.9188	Symphony PLUS program card French
200.9189	Symphony PLUS program card Dutch
200.9190	Symphony PLUS program card Spanish
200.9191	Symphony PLUS program card Italian
200.9192	Symphony PLUS program card Portuguese
200.9193	Symphony PLUS program card Swedish
200.9194	Symphony PLUS program card Norwegian
200.9195	Symphony PLUS program card Danish
200.9196	Symphony PLUS program card Finnish
200.9197	Symphony PLUS program card Polish
200.9198	Symphony PLUS program card Japanese

Symphony Standard program card

Article number	Product
800.0540	Symphony Standard program card German
800.0541	Symphony Standard program card English
800.0542	Symphony Standard program card French
800.0543	Symphony Standard program card Italian
800.0544	Symphony Standard program card Dutch
800.0545	Symphony Standard program card Swedish
800.0546	Symphony Standard program card Danish
800.0547	Symphony Standard program card Finnish
800.0548	Symphony Standard program card Norwegian
800.0549	Symphony Standard program card Polish
800.0550	Symphony Standard program card Spanish
800.0551	Symphony Standard program card Portuguese
800.0552	Symphony Standard program card Greek
800.0553	Symphony Standard program card Hungarian
800.0554	Symphony Standard program card Czech

* Articles may not be available in all countries. For further information about Medela products, visit www.medela.com and look for location finders in your country.

11. Accessories list*

48

Disposable single-use pump sets and bottles

Art. no.	Product
008.0397	Sterile Single-Use Symphony Pump Set size M (24 mm)
008.0398	Sterile Single-Use Symphony Pump Set size L (27 mm)
008.0399	Sterile Single-Use Symphony Pump Set size XL (30 mm)
200.2110	Colostrum Container, set of 2 disposable bottles 35 mL with lids
200.2957	Ready-to-Use Single-Use Bottle, 80 mL, with lid
200.2956	Ready-to-Use Single-Use Bottle, 150 mL, with lid
200.4802	Ready-to-Use Single-Use Bottle, 250 mL, with lid
008.0297	Sterile Single-Use Bottle, 80 mL, with lid
008.0304	Sterile Single-Use Bottle, 150 mL, with lid
008.0305	Sterile Single-Use Bottle, 250 mL, with lid

Disposable one-day pump sets

Art. no.	Product
008.0276	Ready-to-Use One-Day Symphony Pump Set, size M (24 mm)
008.0277	Ready-to-Use One-Day Symphony Pump Set, size L (27 mm)
008.0278	Ready-to-Use One-Day Symphony Pump Set, size XL (30 mm)
008.0176	Symphony One-Day Pump Set, PersonalFit breast shield size M (24 mm)
008.0177	Symphony One-Day Pump Set, PersonalFit breast shield size L (27 mm)
008.0178	Symphony One-Day Pump Set, PersonalFit breast shield size XL (30 mm)
008.0179	Symphony and Lactina One-Day Pump Set, PersonalFit breast shield size M (24 mm)
008.0180	Symphony and Lactina One-Day Pump Set, PersonalFit breast shield size L (27 mm)
008.0181	Symphony and Lactina One-Day Pump Set, PersonalFit breast shield size XL (30 mm)

Reusable pump sets

Art. no.	Product
800.0557	Symphony reusable pump set in PE-bag, breast shield size M (24 mm)
200.1141	Symphony reusable pump set in PE-bag, breast shield size L (27 mm)
200.1143	Symphony reusable pump set in PE-bag, breast shield size XL (30 mm)

Other accessories

Art. no.	Product
800.0605	Basic Kit without tubing
800.0606	Breast shield (size M) and valve complete
800.0641	Comfort breast shield (size M) and valve complete
200.0571	PersonalFit breast shield size S (21 mm)
800.0668	PersonalFit breast shield size M (24 mm)
800.0711	PersonalFit breast shield size L (27 mm)
800.0712	PersonalFit breast shield size XL (30 mm)
800.0854	PersonalFit breast shield size XXL (36 mm)
800.0710	Breast shield connector
800.0660	Symphony membrane cap
800.0632	Symphony protective membrane
800.0608	Symphony membrane cap and protective membrane
800.0659	Silicone tubing with polygon adapters rolled
800.0622	Valve head
800.0623	Valve membrane
800.0624	Valve complete incl. spare membrane
800.0625	Breast milk bottle 150 mL
800.0627	Lid
800.0628	Disc
800.0629	Cap
600.0832	Mobile stand for Symphony complete
810.0552	Bottle stand for Symphony
600.0973	Hard case for Symphony with strap
600.7073	Hard case for Symphony without strap
600.0816	Symphony card and cord protector
928.1017	Car connection cable for Symphony
600.0387	Test vacuum meter

* Articles may not be available in all countries. For further information about Medela products, visit www.medela.com and look for location finders in your country.

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12. Cleaning – country-specific regulations

The process of reprocessing reusable pump sets, excluding silicone tubing, after use as described in chapters 5.7, 5.8 and 5.9 can differ from the practice in your country. For an overview of the country-specific regulations and guidelines used in hospitals, see the table below.

Country	Manual cleaning (§ 5.7)	Automated cleaning (§ 5.8)	Disinfection in washer-disinfector (§ 5.8)	Sterilisation (§ 5.9)	Remarks
Arab Emirates	√*	√*	√	√	Sterilise by autoclave.
Asia Pacific	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.
Australia	√* (use neutral cleaning agent)	√*	√	√	After each use, clean products and then either disinfect (high level) or sterilise them.
Austria	√**	√**	√	n/a	
Belgium	√**	√**	√	√	
Canada	√**	√**	√	√***	Discard tubing and membranes that are exposed to breast milk. Use a sterile bottle and lid with the pump set.
China (CSSD)	√*	√*	√		If high-level disinfection with an A0-value of 3000, for 1 min. at 90°C, sterilisation is not necessary.
CSSD = Central Sterile Supply Department					
Czech Republic	√*	√*	√	n/a	

* Manual cleaning or automated cleaning: both methods are allowed

** Manual cleaning is possible, but automated cleaning is preferred

*** Recommended, not mandatory

Country	Manual cleaning (§ 5.7)	Auto- mated cleaning (§ 5.8)	Disinfection in washer- disinfector (§ 5.8)	Sterilisation (§ 5.9)	Remarks
France (CCLIN) Manual cleaning with sterilisation	√ (clean directly after use)	n/a	n/a	√	
CCLIN = Centre de Coordination de Lutte contre les Infections Nosocomiales					
France (CCLIN) Disinfection in washer- disinfector, no sterilisation	n/a	√	√	n/a	After each use.
France (OMBREL)	√*	√*	√	√	
OMBREL = Organisation Mamans Bébé de la Région Lilloise					
France (SFHH) Premature infants	√**	√**	√	√	If possible, clean directly after use.
SFHH = Société Française d'Hygiène Hospitalière					
France (SFHH) Full-term infants	√**	√**	√	n/a	If possible, clean directly after use.
Germany	√**	√**	√	n/a	Disinfection: use an antiviral agent.
Indonesia	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.
Italy	√*	√*	√	√	
Malaysia	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.

12. Cleaning – country-specific regulations

Country	Manual cleaning (§ 5.7)	Auto- mated cleaning (§ 5.8)	Disinfection in washer- disinfector (§ 5.8)	Sterili- sation (§ 5.9)	Remarks
The Netherlands	√*	√*	√	n/a	The reusable pump set is a single user product.
Philippines	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.
Poland	√**	√**	√	√	
Portugal	√**	√**	√	√	
Russia	√*	√*	√	n/a	Disinfection with boiling water is recommended. Chemical disinfection is also allowed.
Sweden	√**	√**	√	n/a	
Switzerland	√**	√**	√	n/a	If sterilisation is desired, it is recommended to sterilise in the autoclave at 134 °C for 18 minutes
Singapore	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.
Spain	√*	√*	n/a	n/a	Clean as soon as possible after use.
Thailand	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.
UK	√* (use a washing-up liquid)	√*	√***	n/a	It is not necessary to disinfect the tubing, as it does not come into contact with milk. Each mother has her own tubing, which she must keep clean and dry.
Vietnam	√**	√**	√	√***	Perform a pre-clean on the spot immediately after use.

* Manual cleaning or automated cleaning: both methods are allowed

** Manual cleaning is possible, but automated cleaning is preferred

*** Recommended, not mandatory

13. EMC/technical description

The Symphony breast pump needs special precautions regarding EMC (Electromagnetic Compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use. Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, and walkie-talkies can affect the Symphony electric breast pump and should be kept at a distance of at least 1.0 m from the equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

! **Note** – This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- I Reorientate or relocate the receiving antenna.
- I Increase the distance between the equipment and receiver.
- I Connect the equipment to a different outlet on the circuit than that to which the receiver is connected.
- I Consult the dealer or an experienced radio/TV technician for help.

13. EMC/technical description

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 1)

Electromagnetic emissions

The Symphony electric breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Symphony electric breast pump should ensure that it is used in such an environment.

Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Symphony electric breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The Symphony electric breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warning – The Symphony electric breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Symphony electric breast pump should be monitored to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 2)

Electromagnetic immunity

The Symphony electric breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Symphony electric breast pump should ensure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input / output lines	± 2 kV for power supply lines ± 1 kV for input / output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	<5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Symphony electric breast pump requires continued operation during mains power interruptions, it is recommended that the Symphony electric breast pump be powered from an uninterruptible power supply or a battery.

13. EMC/technical description

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
--	-------	--------	---

Note: U_T is the a.c. mains voltage prior to application of the test level

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 4)

Electromagnetic immunity

The Symphony electric breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Symphony electric breast pump should ensure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	10 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Symphony electric breast pump, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 0.35\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Symphony electric breast pump is used exceeds the applicable RF compliance level above, the Symphony electric breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Symphony electric breast pump.

^b Field strengths should be less than 10 V/m over the 150 kHz to 80 MHz frequency range.

13. EMC/technical description

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 6)

Recommended separation distances between portable and mobile RF communications equipment and the Symphony electric breast pump

The Symphony electric breast pump is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Symphony electric breast pump can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Symphony electric breast pump as recommended below, according to the maximum output power of the communications equipment.


Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = 0.35\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.04	0.12	0.23
0.1	0.11	0.37	0.74
1	0.35	1.2	2.3
10	1.11	3.7	7.4
100	3.50	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.


Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

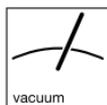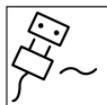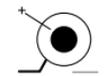
Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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