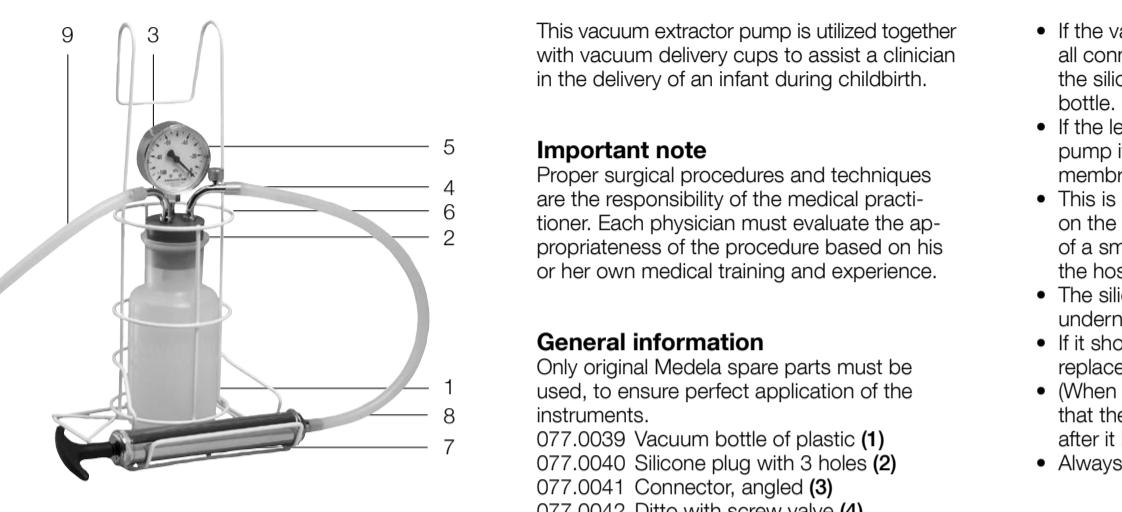
VACUUM
EXTRACTOR PUMPVACUUM
EXTRACTOR PUMPVAKUUM
EXTRAKTOR PUMPEPOMPE D'EXTRACTION
SOUS VIDEPOMPA DI ESTRAZIONE
PER VUOTOVACUÜM-
EXTRACTORPOMPVAKUUM-
EKSTRAKTORPUMPE

GENERAL INFORMATION

EN Safety rules

DE ALLGEMEINE INFORMATIONEN
SicherheitsregelnFR GÉNÉRALITÉS
Règles de sécuritéIT INFORMAZIONI GENERALI
Norme di sicurezzaNL ALGEMEEN
VeiligheidsvoorschriftenSV SÄKERHETS-
FÖRESKRIFTERDA SIKKERHEDS-
FORSKRIFTER

CE 0123

Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.comInternational Sales
Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
Phone +41 562 51 51
Fax +41 562 51 00
info@medela.ch
www.medela.comGermany
Medela Medizintechnik
GmbH & Co. KG
Postfach 145, 8830 Erding
Phone +49 83 11 97 59 00
Fax +49 83 11 97 59 99
info@medela.de
www.medela.deSwitzerland
Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
Phone +41 562 53 352
Fax +41 562 51 00
contact@medela.ch
www.medela.ch

© Medela AG 177/022/2016/09-X

This vacuum extractor pump is utilized together with vacuum delivery cups to assist a clinician in the delivery of an infant during childbirth.

Important note
Proper surgical procedures and techniques are the responsibility of the medical practitioner. Each physician must evaluate the appropriateness of the procedure based on his or her own medical training and experience.

General information
Only original Medela spare parts must be used, to ensure perfect application of the instruments.

77.0003 Vacuum bottle of plastic (1)
77.0040 Silicone plug with 3 holes (2)
77.0044 Convex, angled (3)
77.0042 Valve with valve (4)
77.0045 Vacuum-gauge with straight connector (5)
77.0046 Plastic covered basket for bottle (6)
77.0047 Vacuum pump (7)
77.0054 Vacuum tubing, 50 cm long, for Malmstrom model (8)
77.0055 Silicone tubing, 120 cm long, for Malmstrom model (9)

Troubleshooting

Description

The cup does not get attached, or it slips, or becomes completely detached from the model.

Causes

- Leaks or other faults in the apparatus (faulty pump, worn or damaged tubing, loose connections, detited rim of cup).

- Soft cups being bent and snap.

- Only pump valve is tight when using a New Generation cup.

- Large caput (caput deeper than cup).

Management

- If the cause is obvious (e.g. loose connection), make quickly correctable, correct and start again.

- If the cause is not obvious, try again with the cup in a slightly different position (further back if possible).

- In either case, always pull with the pump valve if it is essential, or if the second attempt with the vacuum extractor fails and if the necessary pre-requirements for using forceps are present.

Safety rules

If the vacuum gauge has experienced a strong impact (e.g. if it fell on the floor), it may not be used again after the functional and safety tests have been verified.

Do not use if:

- If the device is damaged.

- If the device shows clear safety defects.

- Wall suction is not to be used.

- Vacuum extraction must only be performed by trained and experienced clinicians.

- The neonatal care staff must be aware of the indications, contraindications and precautions for assisted delivery and must attach the same importance of complications of vacuum assisted delivery.

- Operative vaginal delivery should not be performed while the probability of success is very low.

- Endotracheal intubation of extreanous tissues and ensure proper placement of cup.

- Attention: Never apply the cup to any portion of the infant's face as this can cause fetal trauma.

- Insertion should be performed carefully avoiding infections, maternal or fetal trauma end maternal discomfort.

- The head, not just the scalp, must descend with each pull.

- Pull only during a contraction.

- Always apply traction at an angle according to the axis of the cup to remain in-line or orthogonal.

- Never apply rocking movements or torque to the device; only steady traction should be used.

- Before using vacuum extraction if there is evidence of significant scalp trauma or damage.

- The head must be delivered, or almost completely delivered, with no more than three pulls.

- The cup must be completely delivered within 15–20 minutes of first applying the cup.

- Never exceed 10 minutes of cumulative traction time.

- Never exceed maximum recommended vacuum –80 kPa (–600 mmHg) or –pop offs (2) – scalp trauma can occur.

- Discontinue vacuum extraction if the vacuum cup becomes disengaged or pops off.

- Do not pull three times, if the second pull fails to provide good descent.

Cleaning

The silicon bottle, gauge, silicone plug and its two metal pipes, pump and tubing must not be sterilized. The bottle should be kept clean by washing with soap and water.

Routine Control

Routine testing of the pump

Periodically (depending on the frequency of use) check the accuracy of the vacuum gauge.

The maximal tolerance is +/- 2.5 kPa

compared to a reference vacuum gauge.

The pump and the related parts should be tested before being used for the first time and then every two or three days.

• Firmly screw down the cap of the vacuum release valve.

• Have a helper place a moist fingertip tightly against the mouth of the hose connector elbow pipe.

• Make a few strokes with the vacuum pump to draw the vacuum to about –40 kPa (–300 mmHg).

• The vacuum should rise quickly and remain stable until released (by removal of the finger, or by opening the vacuum release valve).

If the vacuum is seen to fall rapidly, check all connections for leaks; remove and wet the silicone rubber plug and replace it in the bottle.

If the leak is still not corrected, check the pump itself, particularly the thin silicone membrane in the front of the pump.

This is easily done by unscrewing the nut on the pump handle (loosen it by means of a small wrench or pliers and drawing out the hose connector piece).

The silicone membrane will be seen just underneath.

If there shows signs of wear or perforation,

then a new one.

When the valve is at fault, it will be noted that the pump handle will not be drawn in after it has been pulled out.

• Always keep at least one spare membrane.

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77.0041 Convex, angled (3)

77.0042 Valve with valve (4)

77.0045 Vacuum-gauge with straight connector (5)

77.0046 Plastic covered basket for bottle (6)

77.0047 Vacuum pump (7)

77.0054 Vacuum tubing, 50 cm long, for Malmstrom model (8)

77.0055 Silicone tubing, 120 cm long, for Malmstrom model (9)

Troubleshooting

Description

The cup does not get attached, or it slips, or becomes completely detached from the model.

Causes

- Leaks or other faults in the apparatus (faulty pump, worn or damaged tubing, loose connections, detited rim of cup).

- Soft cups being bent and snap.

- Only pump valve is tight when using a New Generation cup.

- Large caput (caput deeper than cup).

Management

- If the cause is obvious (e.g. loose connection), make quickly correctable, correct and start again.

- If the cause is not obvious, try again with the cup in a slightly different position (further back if possible).

- In either case, always pull with the pump valve if it is essential, or if the second attempt with the vacuum extractor fails and if the necessary pre-requirements for using forceps are present.

Safety rules

If the vacuum gauge has experienced a strong impact (e.g. if it fell on the floor), it may not be used again after the functional and safety tests have been verified.

Do not use if:

- If the device is damaged.

- If the device shows clear safety defects.

- Wall suction is not to be used.

- Vacuum extraction must only be performed by trained and experienced clinicians.

- The neonatal care staff must be aware of the indications, contraindications and precautions for assisted delivery and must attach the same importance of complications of vacuum assisted delivery.

- Operative vaginal delivery should not be performed while the probability of success is very low.

- Endotracheal intubation of extreanous tissues and ensure proper placement of cup.

- Attention: Never apply the cup to any portion of the infant's face as this can cause fetal trauma.

- Insertion should be performed carefully avoiding infections, maternal or fetal trauma end maternal discomfort.

- The head, not just the scalp, must descend with each pull.

- Pull only during a contraction.

- Always apply traction at an angle according to the axis of the cup to remain in-line or orthogonal.

- Never apply rocking movements or torque to the device; only steady traction should be used.

- Before using vacuum extraction if there is evidence of significant scalp trauma or damage.

- The head must be delivered, or almost completely delivered, with no more than three pulls.

- The cup must be completely delivered within 15–20 minutes of first applying the cup.

- Never exceed 10 minutes of cumulative traction time.

- Never exceed maximum recommended vacuum –80 kPa (–600 mmHg) or –pop offs (2) – scalp trauma can occur.

- Discontinue vacuum extraction if the vacuum cup becomes disengaged or pops off.

- Do not pull three times, if the second pull fails to provide good descent.

Cleaning

The silicon bottle, gauge, silicone plug and its two metal pipes, pump and tubing must not be sterilized. The bottle should be kept clean by washing with soap and water.

Routine Control

Routine testing of the pump

Periodically (depending on the frequency of use) check the accuracy of the vacuum gauge.

The maximal tolerance is +/- 2.5 kPa

compared to a reference vacuum gauge.

The pump and the related parts should be tested before being used for the first time and then every two or three days.

• Firmly screw down the cap of the vacuum release valve.

• Have a helper place a moist fingertip tightly against the mouth of the hose connector elbow pipe.

• Make a few strokes with the vacuum pump to draw the vacuum to about –40 kPa (–300 mmHg).

• The vacuum should rise quickly and remain stable until released (by removal of the finger, or by opening the vacuum release valve).

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